



Ardmore Healthcare Single Use G.I. Manometry Catheters Instructions for Use

Please read and understand all information contained in these Instructions for Use before use of Single Use G.I. Manometry catheters.

The user should always confirm that the marker accuracy stated below is suitable for the procedure being undertaken.

- Datum hole to 1st Marker = +/- 1.0mm
- Line spacing of 10mm (1cm) = +/- 1.0mm

Single patient use only - DO NOT STERILISE

Indications for Use:

G.I. Manometry catheters are intended for water-perfused manometry of the G.I tract. It is indicated for use in the diagnosis and treatment of gastrointestinal motility disorders to record and analyse the following data:

- Motility
- Swallowing
- Colonic
- Antroduodenal
- Anorectal
- Tracheal

Contraindications:

The Ardmore Healthcare Single Use G.I. Manometry Catheter is contraindicated for patients presenting with

- Local infection of the G.I. tract.
- Physical obstruction of a natural orifice for which the product is intended.

Warnings & Precautions:

- For use only by qualified medical professional.
- Do not use in contact with damaged tissue.
- Do not stretch or pull the catheter.

Balloon Capacity:

The clinician must determine the appropriate and safe level of inflation for each patient.

1. Refer to the instructions of your Solar Manometry System for the use, care, maintenance and calibration of your equipment.
2. Open packaging and remove catheter.
3. If necessary, purge air from pump.
4. Attach the luer connector of the catheter to the luer connectors on the transducers of the perfusion pump.
5. Introduce the catheter at the desired location in the G.I. tract and perform manometric studies following your Hospital/Clinic protocol.

Catheter Disposal:

After withdrawal, remove and dispose of the catheter safely following your Hospital/Clinic protocol.

Storage:

Store at room temperature away from direct sunlight.

ARDMORE HEALTHCARE LTD

25 South Road, Amersham, Buckinghamshire HP6 5LU

Tel: +44 (0) 1494 721820 Fax: +44 (0) 1494 400330

Email: info@ardmorehealthcare.com